## INSTRUCTIONS FOR USE

### INTENDED USE

Sperm-Chex® is an assayed control intended for monitoring sperm counts performed manually using commercially available counting chambers to validate quantification of sperm in seminal fluids.

### SUMMARY AND PRINCIPLES

Microscopic evaluation of semen includes the determination of sperm concentration. The sperm count can provide valuable information regarding infections, stress, and fertility. When handled like a patient sample, Sperm-Chex controls will provide sperm counts within the expected ranges indicated on the assay sheet.

### REAGENTS

Sperm-Chex control contains stabilized mammalian sperm in a preservative medium.

### PRECAUTIONS

1. For in Vitro Diagnostic Use.
2. Product contains dilute formalin.
3. This product should not be disposed in general waste, but should be disposed with infectious medical waste. Disposal by incineration is recommended.
4. SDS can be obtained at [streck.com](http://streck.com), by calling 800-843-0912, or by calling your local supplier.

### STORAGE AND STABILITY

Sperm-Chex control is stable through the expiration date when stored at 2 °C to 10 °C. Cap the vial tightly and store upright. After opening, Sperm-Chex is stable throughout the open-vial dating, as indicated on the assay sheet, when stored at 2 °C to 10 °C.

### INDICATIONS OF PRODUCT DETERIORATION

Inability to obtain expected value for sperm counts may indicate product deterioration. If the recovered values are not within the expected ranges:

1. Review the control product package insert Instructions For Use and Limitations sections.
2. Check the expiration date of Sperm-Chex. Discard outdated product.
3. Assay an unopened vial of Sperm-Chex. If the values are still outside the expected range, contact Streck Technical Services at 800-843-0912 or [technicalservices@streck.com](mailto:technicalservices@streck.com).

### INSTRUCTIONS FOR USE

1. Remove the controls from the refrigerator. It is not necessary to warm the controls to room temperature before using.
2. To mix:  
**For a video demonstration, visit [streck.com/mixing](http://streck.com/mixing).**
  - a. Pipette any liquid from the cap and add to the contents of the vial before mixing. Do not invert vial during mixing.
  - b. **Vortex on medium speed for 3 to 5 pulses of 3 to 5 seconds each, verifying that a vortex is obtained. The pellet on the bottom of the vial should be dislodged and the clear liquid may turn turbid during the mixing process. Do not mix the product in any other manner. Other mixing methods will not properly re-suspend the sperm.**
3. Any isotonic solution may be used to dilute the sample prior to loading a counting chamber. (**NOTE: No dilution is needed for use with the Makler counting chamber.**)
4. Make dilutions using a calibrated micropipettor, following pipet manufacturer's recommended pipetting techniques. The vial must be closed immediately after sampling is complete. Care should be taken to avoid cross-contamination of the control. Wipe the threads of both the vial and the cap before replacing the cap.
5. Mix dilutions thoroughly before loading the chamber. No dilutions for the Makler counting chamber.
6. Perform the count according to the hemocytometer package insert, Makler counting chamber instructions or your established laboratory protocols. Count only complete sperm with head, mid-piece and tail.
7. After sampling, return the vial to refrigeration for maximum open-vial stability.

### LIMITATIONS

1. Analysis of Sperm-Chex cannot be performed on automated cell counting systems.
2. Sperm-Chex control is assayed for sperm count only and cannot be used for morphology, motility, or viability testing.

### EXPECTED RESULTS

The assigned values were determined by replicate analyses of hemocytometer counts and Makler chamber counts. The assigned values of the sperm counts are based on the average number of sperm per ml counted in the four large corner squares on both sides of the Neubauer improved hemocytometer counting chamber and in a strip of 10 squares for the Makler counting chamber. Assay range values were established based on  $\pm 3SD$ . Upon receipt of a new control lot, it is good laboratory practice that an individual laboratory establish its own mean and limits. However, the control means established by the laboratory should fall within the expected range specified for the control.

### PERFORMANCE CHARACTERISTICS

The assigned values are presented as a mean and a recovery range. The mean assay values are derived from repetitive testing on multiple vials of product using Neubauer improved hemocytometers and Makler counting chambers, which are operated and maintained according to the manufacturer's instructions. The recovery ranges are intended to reflect laboratory variability. Always perform quality control according to good laboratory practice, laboratory director's requirements, and any regulatory or accredited requirements.

### QUALITY CONTROL PROGRAM

Streck offers *STATS*®, an interlaboratory quality control program, to all customers at no charge. For more information, contact the *STATS* Department at 800-898-9563 or [statsdata@streck.com](mailto:statsdata@streck.com). Additional information can be found at [streck.com](http://streck.com).

### ORDERING INFORMATION

Please call our Customer Service Department at 800-228-6090 for assistance. Additional information can be found online at [streck.com](http://streck.com).

### Rx Only

## GLOSSARY OF SYMBOLS

See the Instructions (IFU) tab under Resources on the product page at [streck.com](http://streck.com).

See [streck.com/patents](http://streck.com/patents) for patents that may be applicable to this product.

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